

# Sweet relief from tooth sensitivity.

Proven long-term results with instant  
patient comfort in only 30 seconds with

**Shield Force® Plus.**



 Tokuyama



# Introducing Shield Force<sup>®</sup> Plus Desensitizer

By utilizing Tokuyama's double-block technology, Shield Force<sup>®</sup> Plus quickly and effectively seals the dentin tubules, offering instant and long-term protection for tooth sensitivity. Its simplified one-step procedure takes only 30 seconds from start to finish, and its green color enables you to accurately place the solution on the sensitive area. Indicated for direct or indirect restorations, Shield Force<sup>®</sup> Plus is versatile, allowing your patients to finally receive the relief they deserve!


## A patented technology with a one-two punch

The secret sensitivity-fighting compound in Shield Force<sup>®</sup> Plus is its **double-block** technology. By sealing the dentin tubules with resin tags that permeate 50µm deep, it quickly relieves sensitivity. When cured, it forms a long-lasting durable shield that blocks sensitivity and also prevents loss of dentin due to abrasion.

## Thin coating layer with deep penetration

At first application, Shield Force<sup>®</sup> Plus immediately reduces painful sensitivity. It offers a uniform thin coating layer as well as deeper penetration into the dentin tubules, forming cross-links to apatite on the tooth surface and calcium ions coming from tooth structure.

## Sealed with resin tags and layers

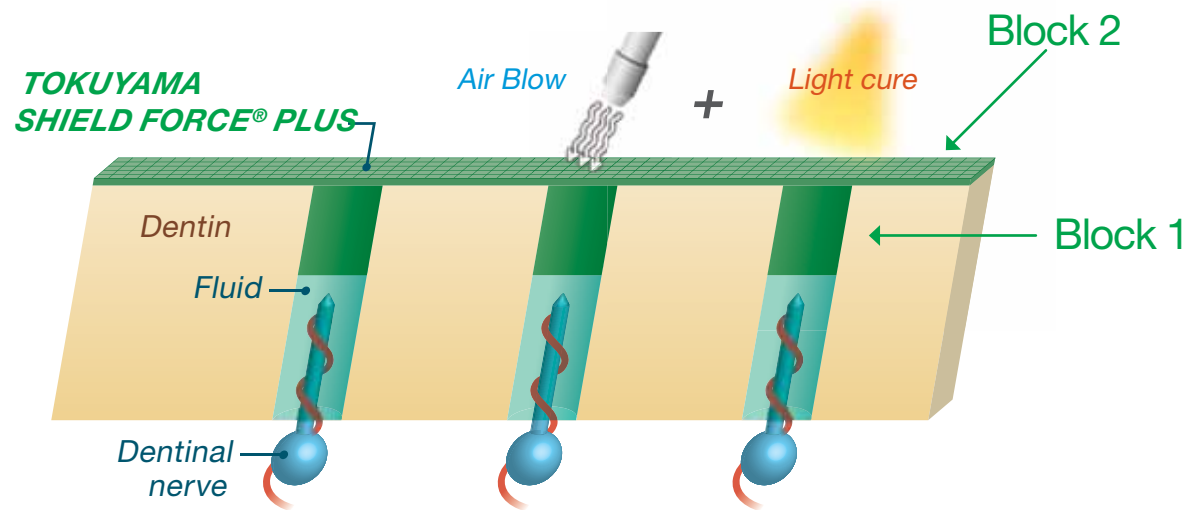


Tokuyama's double-block effect seals the dentin tubule to immediately relieve pain, and after light curing, creates a strong durable coating for long-term results.

## Superior Long-Term Results


With its durable seal and long-term results, Shield Force<sup>®</sup> Plus outlives and outshines most desensitizing competitors.

### Dentinal tubules sealing and durability

Tokuyama's Shield Force<sup>®</sup> Plus layer remains on the dentin surface after testing. Thermacycle test (10,000 thermal cycles) 4°C – 60°C

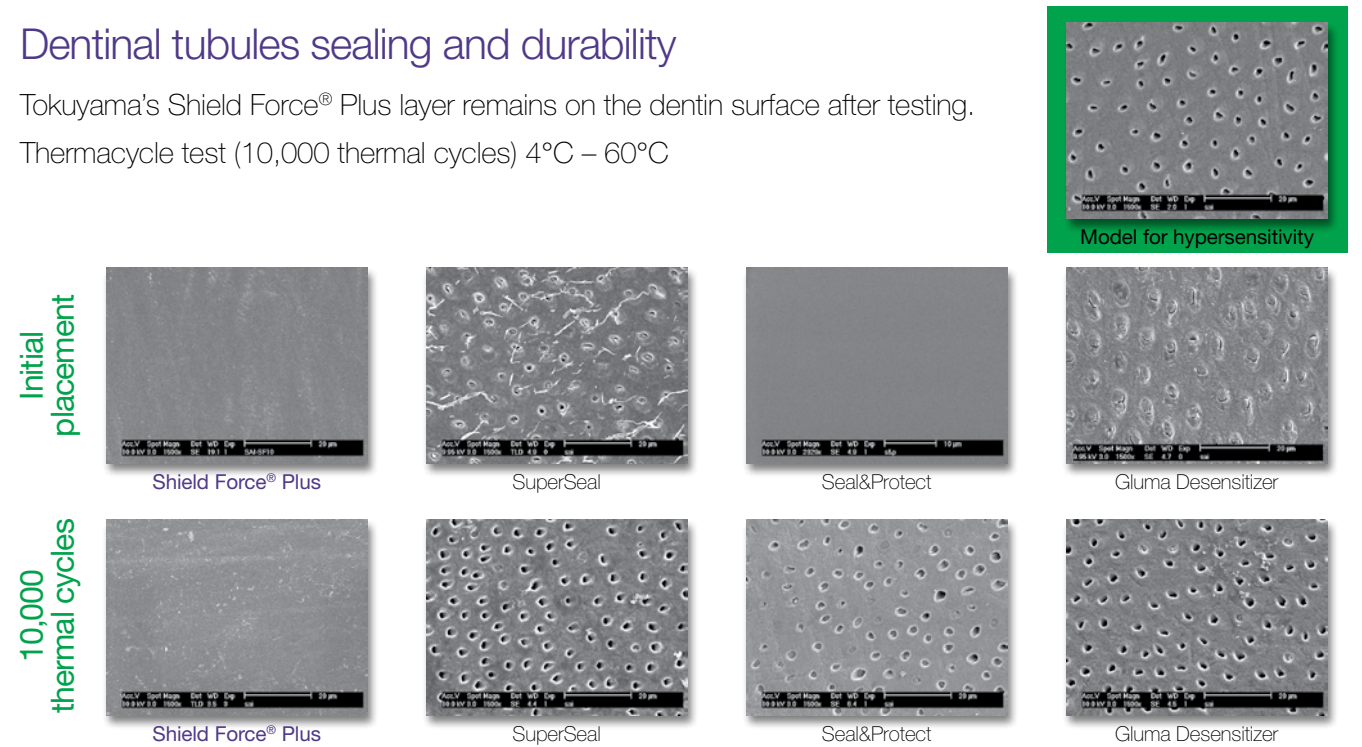

### High wear resistance to toothbrush abrasion

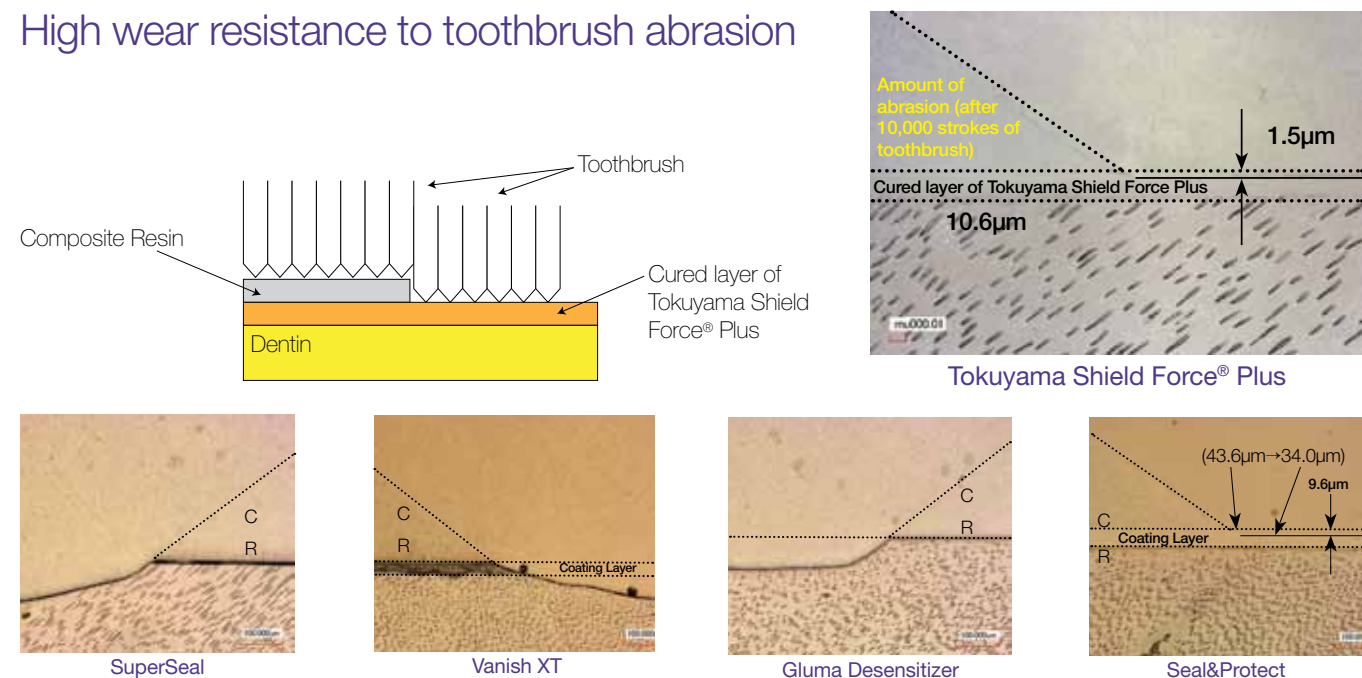

## Easy application

Shield Force® Plus has a simplified application process which reduces procedure time, and is very easy to administer. The procedure requires no rinsing, no mixing, and only takes 30 seconds from start to finish. Shield Force® Plus also goes on green to accurately place on designated areas, yet dries clear for superior results.



## Tokuyama Shield Force® Plus Kit

Item # 15111

- Bottle 3mL x1
- Disposable applicator (superfine) x25
- Dispensing well x1

## Tokuyama Shield Force® Plus Refill

Item # 15121

- Bottle 3mL x1

## About Tokuyama

Tokuyama draws on nearly a century of accomplishments and knowledge in the dental industry. Operating on a global scale, our mission is to listen to your needs, incorporate your voice into the development of innovative products, and ensure our products meet the highest standards of quality. Tokuyama is a steward of cutting-edge technology and specialty products that improve your life and the lives of your patients.



[www.tokuyama-us.com](http://www.tokuyama-us.com)

1-877-378-3548

Tokuyama Dental America Inc. • 740 Garden View Ct., Suite 200 • Encinitas, CA 92024

*Innovating Tomorrow's Dentistry, Today.*