

## SECTION VI : ACCIDENTAL RELEASE MEASURES

None

## SECTION VII : HANDLING AND STORAGE

Spill Management: Use absorbent to collect the material. Wash contaminated surfaces with Soap and water

## SECTION VIII : EXPOSURE CONTROLS/PERSONAL PROTECTION

Respiratory: None

Eye Protection: Safety goggles

Gloves: Surgical, rubber/PVC gloves

Other Clothing and Equipment: Face Mask

Ventilation: None required, local exhaust recommended

## SECTION IX: PHYSICAL AND CHEMICAL PROPERTIES

Vapor Pressure: NA

Vapor Density: NA

Evaporation Rate: NA

Solubility in Water: Insoluble

Boiling Point: ND

Percent Volatile: 2

Appearance and Odor: Silicon (various different colors) no odor

## SECTION X: STABILITY AND REACTIVITY

Stable (x) Unstable ( )

Conditions to Avoid: Heat in excess of 40°C, direct sunlight or intense light.

Incompatibility: Free radical initiators, oxidizing agents

Hazardous Decomposition Products: Acrylic smoke

Hazardous Polymerization: May occur ( ) Will not occur (x)

## SECTION XI: TOXICOLOGICAL INFORMATION

Carcinogens: None known

## SECTION XII : ECOLOGICAL INFORMATION

This material contains hazardous components. Allow materials to cure prior to disposal.

## SECTION XIII : DISPOSAL CONSIDERATIONS

Dispose of safely in accordance with local, state, and federal regulations.

## SECTION XIV : TRANSPORT INFORMATION

Stable under normal conditions of use, transportation, and storage.

## SECTION XV: REGULATORY INFORMATION

510k#: K951075

## SECTION XVI : OTHER INFORMATION

None

The data and information given in this msds are accurate on the date of preparation. It does not indicate any warranty or representation. We disclaim all liability relating to use of this material since this is beyond our control.



3420 FOSTORIA WAY STE. A-200 SAN RAMON, CALIFORNIA 94583 USA  
PHONE 800/827-7940 FAX 925/973-0764

0419 REV J

DANVILLE

IMPRESSION MATERIAL

# Star VPS™



## INSTRUCTIONS

Star VPS offers a variety of formulations. Each formulation is the result of extensive research to provide dependable results, ease of use, and improved clinical performance.

Star VPS is odorless, tasteless and immersible in disinfectants. It offers dimensional stability, tear resistance, and accuracy of impression.

Star VPS	Color	Working Time At 72°F	Setting Time At 72°F
Light SF	Magenta	2.5 Minutes	4.5 Minutes
Light SF Fast	Purple	30 Seconds	1.75 Minutes
Monophase	Violet	30 Seconds	1.75 Minutes
Heavy Flex	Dark Blue	2.5 Minutes	4.5 Minutes
Heavy Stiff Bite	Light Blue	20 Seconds	50 Seconds
Vanilla Flavored Stiff Bite	Off White	20 Seconds	50 Seconds
Clear Bite	Clear	25 Seconds	90 Seconds
Putty	Lime Green	1.5 Minutes	4.5 Minutes
Ultra Light	Teal Blue	2.5 Minutes	4.5 Minutes
Ultra Heavy	Yellow	2.5 Minutes	4.5 Minutes

## MIXING INSTRUCTIONS - CARTRIDGE

1. Insert cartridge into gun, remove twist-off cap, and extrude a 1/4 inch of material, while checking for even flow. Discard the dispensed material and wipe end of cartridge clean.
2. Attach an auto mix tip and squeeze the cartridge handle with smooth, even pressure.
3. Do not remove the automix tip after use. The used tip serves as a convenient seal until next use.

## MIXING INSTRUCTIONS - PUTTY

1. Scoop equal amounts of the putty and catalyst from their respective jars. Two scoops are provided.
2. Mix (kneading by hands) until an even, streak free color is obtained (approximately 45 seconds).

Note: Powder from glove surfaces can inhibit setting.

**CLOSED BITE IMPRESSIONS** (Use Light SF and Heavy Flex or Light SF Fast and Monophase)

1. Select an appropriate tray such as Premier Triple Tray. (**Technique Tip:** *It is best to avoid metal trays "bite trays" if a second pour is made in the laboratory. Metal trays tend to distort when the lab pries the first cast out of the impression, precluding an accurate second pour.*)
2. The impression should be taken using two viscosities simultaneously: Heavy Body Flex (or Monophase) in the tray for dimensional stability and a wash of Light SF or Light SF Fast for detail. The key to this procedure is to syringe Light SF (or Light SF Fast) onto clean, dry teeth, then blow thin air until only a thin film remains. If a blank area remains, dry, syringe, and blow again, until only the thin film remains. Add Light SF or Light SF Fast to cover tooth, then seat tray.
3. Have patient close onto a tray of Heavy Flex (or Monophase) and guide patient into a CO closure. It is important to rehearse the proper closure beforehand. NOTE: Putty should never be used for this procedure. It is too viscous, and induces elastic distortion.
4. Water wash and dry the impression after removal from the mouth.

**SEPARATE FULL ARCH "PUTTY/WASH" IMPRESSIONS**

(Use Light SF or Light SF Fast and Putty)

Creating accurate impressions using putty requires a dual set technique. The putty is allowed to fully polymerize in the metal or plastic stock tray before the wash step. NOTE: When using a custom tray made from a preliminary impression, use adhesive on the tray and allow to dry for 5 minutes. Putty is not needed and Light SF or Light SF Fast is all that is needed.

1. Before cutting the prep, make a putty impression, leaving room around the teeth for the wash. Leaving a space for the wash is achieved by simply placing a plastic film (such as a section of a baggie or Reynolds Wrap) over the putty before seating the tray. IMPORTANT: Some plastic wraps will inhibit the set; test before use.
2. Seat the tray with the putty, let polymerize, then remove tray and await prep.
3. Use Light SF or Light SF Fast to take the final impression. Remove plastic film from the tray. Syringe Light SF or Light SF Fast onto clean dry teeth. Blow off with air until only a thin film remains. Repeat to cover any blank spots. The needle attachment for the small mixing tip is very handy for inlay, onlay and deep margins.
4. Syringe Light SF or Light SF Fast into putty impression and seat.
5. Remove after polymerization, wash and dry. IMPORTANT: Avoid simultaneous putty/wash set as putty is elastic and may cause distortion.

**MONOPHASE IMPRESSION** (Use Monophase)

Single material impressions can be used where Light SF is not required for high flow. Monophase has a rapid set and fine texture, and is an ideal material to use for simple closed bite impressions as well as a preliminary for Turbo Temp™ temporary crown and bridge material.

1. Syringe Monophase around clean, dry teeth. Syringe into sideless tray.
2. Have the patient close until polymerized. Remove, wash and dry.

**SEPARATE FULL ARCH DUAL VISCOSITY IMPRESSION**

(Use Ultra Light and Ultra Heavy)

1. Apply a thin layer of tray adhesive and allow it to dry at least 5 minutes.
2. Automix. Using a cartridge of Ultra Heavy and a cartridge of Ultra Light, bleed a small amount of material from each syringe and discard. Attach a mixing tip to each cartridge and mix simultaneously. Fill the tray with the Ultra Heavy material and either front-load an intraoral syringe with the Ultra Light material or attach the intraoral tip and inject directly around the clean, dry preparations. The working time is 2 minutes 30 seconds.

3. Slowly but firmly seat the tray in the mouth.
4. After 4 minutes and 30 seconds from the start of the mix, remove the tray from the mouth.

**BITE REGISTRATION** (Use Heavy Stiff Bite or Clear Bite)

Star VPS Heavy Stiff Bite and Clear Bite are dimensionally very stable. Use where flexibility is not required.

**ADDITIONAL NOTES:**

- Star VPS materials should be brought to room temperature prior to use. Exposure to prolonged temperatures above 77°F can be damaging. Store at room temperature.
- Star VPS materials are compatible with all other vinyl polysiloxane materials.
- Powder from gloves can impair set. Sample test is suggested. Keep putty jars closed when not in use.
- High viscosity materials used alone are not suitable for detailed impressions.
- Light Body Black impression materials used alone can flex excessively and may result in distortion.
- Procedures and techniques prepared courtesy of Raymond Bertolotti, DDS, PhD. For further information, please contact 5th Quarter Seminars at (510) 483-2411, FAX (510) 652-8729. [www.adhesion.com](http://www.adhesion.com)

**MATERIAL SAFETY DATA****SECTION I - PRODUCT IDENTIFICATION**

Company Name: Danville Materials, Inc.  
3420 Fostoria Way, Ste A-200  
San Ramon, CA 94583  
Phone: (800) 827-7940  
Fax: (925) 973-0764  
Prepared: November 16, 2012

**SECTION II : HAZARD(S) IDENTIFICATION**

OSHA Permissible Exposure Limits: None  
Other Exposure Limit Used: None  
ACGIH Threshold Exposure Limit: None  
Chronic, Other: None

**SECTION III : COMPOSITION/INFORMATION ON INGREDIENTS**

Hazardous Component  
Mixture of Polydimethylsiloxane, Silica and Paraffin  
Chemical Family: Silicon  
No known hazardous components

**SECTION IV : FIRST-AID MEASURES**

Eye Contact: Flush eyes with large amounts of water, consult a physician  
First Aid Procedures: For Skin - Wash off infected area with soap and water.  
For Ingestion - Seek medical advice, carry container with label and MSDS.  
Effects of over Exposure: N/A

**SECTION V: FIRE-FIGHTING MEASURES**

Flash Point: 485°F (252 °C) closed cup – DIN 51755  
Extinguishing Media: Carbon dioxide, Water  
Special Fire Fighting Procedures: None  
Flammable limits: ND  
Unusual Fire and Explosion Hazards: Irritating and/or toxic gases and aerosols may be present from the decomposition/combustion of product.